



# Capsule Cleaning Instructions<sup>1</sup>

Prior to downloading data from the used CapsoCam<sup>®</sup> Plus capsule, it must be cleaned and disinfected.



1

## Clean the capsule using ENZOL<sup>®</sup> Enzymatic Cleaner or equivalent according to the manufacturer's instructions<sup>2</sup>:

**1 oz. (2 TBSP)** / **1 gal.**

Add one ounce (2 tablespoons) of ENZOL<sup>®</sup> Enzymatic Detergent per gallon of water.

Soak capsule in sufficient solution to submerge completely (at least 5 cc). Organic matter should be removed. A minimum soak of one minute is recommended.

**1 min.**

If necessary, mechanically clean capsule, including dimple, with a soft bristle brush.

Thoroughly rinse capsule to remove ENZOL<sup>®</sup> Enzymatic Detergent. Capsule is now ready for high-level disinfection.

Discard ENZOL<sup>®</sup> Detergent after each use or when visibly soiled. Flush used solution down the drain with water or in accordance with local regulations.

2

## Disinfect the capsule using Revital-Ox<sup>™</sup> Resert<sup>®</sup> High Level Disinfectant\* or equivalent and rinse thoroughly, according to the manufacturer's instructions<sup>3</sup>.

Place pre-cleaned, rinsed and dried capsule in 5 cc of undiluted Revital-Ox<sup>™</sup> RESERT<sup>®</sup>

Soak capsule in sufficient solution to submerge completely (at least 5 cc) for a minimum of 8 minutes at ≥ 20°C (68°F). Soaking overnight is acceptable.


**8 min.**

Rinse the capsule with water.

Discard Revital-Ox<sup>™</sup> Detergent after each use. Flush used solution with water or in accordance with local regulations.

3

## Dry capsule completely.\*\*

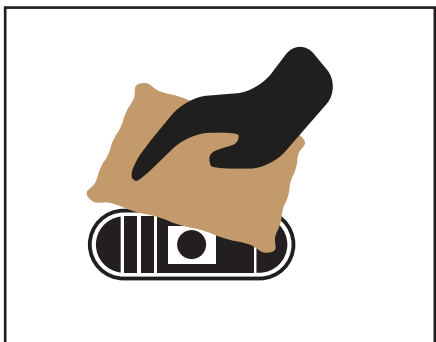

\* Revital-Ox<sup>™</sup> RESERT<sup>®</sup> High Level Disinfectant has demonstrated disinfection efficacy on the presence of organic soil contamination and microbiological burden during reuse and may be reused for up to 21 days. Please see manufacturer's instructions for limitations and indications related to reuse.

\*\* Capsules with remnant moisture may have difficulty downloading data. Therefore, please ensure that the capsule is completely dried before attempting download.

### Sources:

- 1 CapsoVision, Inc., "Instructions for Use, CapsoCam Plus", IFU-2796 US (English).
- 2 Ethicon US, LLC "Enzol Automatic Detergent: Experience the Power of Fast-Acting Clean", AD-50255-01-US\_D, Promotional Data Sheet, 2014.
- 3 Steris Corporation, "Steris Revital-Ox<sup>™</sup> RESERT<sup>®</sup> High Level Disinfectant Technical Data Sheet", 330-300-4455 Data Sheet, January 1, 2016.